

What is the Versa⚡Volt™ power system?

The HT50 Ventilator can be powered by:

- 100 – 240 VAC
- Internal DC battery (up to 10 hours)
- 12 – 30 VDC external battery

from a single power entry module!

- The Versa⚡Volt™ power entry module is completely self-switching and adaptive – with only one connector and no fuses or switches to change.
- This amazingly flexible power entry module automatically recognizes and switches between AC and DC power and between different ranges of AC and DC voltages.
- The internal battery charges from external AC or DC whenever it is plugged in, even when the HT50 is not powered on.



The Versa⚡Volt™ power system on the HT50 gives you the flexibility you need!