


**SUBJECT: Newport HT50 Ventilator Proximal Inline Filter (p/n HT6004701)
is Not Compatible with Isopropyl Alcohol**

The small hydrophobic filter – Proximal Inline Filter (p/n HT6004701) – that is required for use at all times inline with the Newport HT50 Ventilator proximal pressure line is not compatible with Isopropyl Alcohol.

Never allow Isopropyl Alcohol to come in contact with the filter when wiping down the external surface of the ventilator as it will cause the housing of the filter to melt.

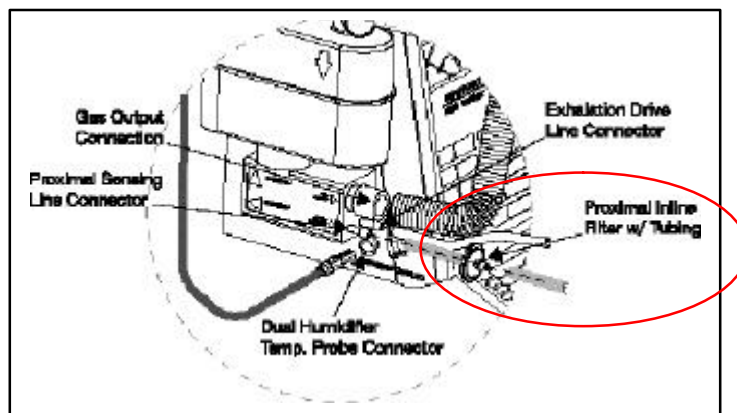

If you have any questions or need additional information please contact Newport Customer Service: 1.949.6423910 ext 282 or email: info@newportnmi.com.